use/install method

- Thank you for purchasing ECE Bedside handrail YC9100.
 - Before use, please read the instructions carefully and use correctly and understand.
- Please keep this manual properly after reading, so that you can consult and refer to it at any time.
- 1. YC9100 This product can help the elderly, pregnant women and other people with mobility difficulties at home to take care of themselves, get up and down the bed no longer use help, the armrest side of the storage bag can be put on the elderly urgent items.
- 2. The surface of the handrail is beige powder treatment, the surface is smooth and delicate, beautiful and durable, high-grade, no corrosion, no rust, no tool installation, no need to fix other screws, no damage to the bed body, the use of more intimate with 6 meters long seat belt, suitable for different widths of the bed, the pressure pad tube under the mattress is flat tube, the pressure is more firm
- 3. YC9100 is suitable for the elderly, the sick, pregnant women and other people who are not convenient to get up.



Product model: YC9100

Product name: bedside

handrail

Net weight: 2.0KG

Gross weight: 3.0KG

Note: Manual measurement error 1-5cm



Figure 1. The package contains the following items:

(armrest, 1 pad flat tube, 1 storage bag and 1 seat belt).

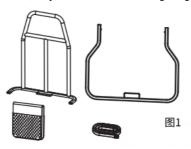


Figure 2. Press the copper marbles on both sides of the flat tube with both hands at the same time, align the flat tube holes of the armrest frame, and press the flat tube into the flat tube inserted into the armrest frame until the copper marble pops out of the hole, so as to complete the work.



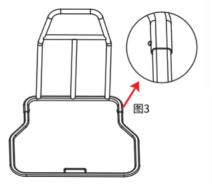


Figure 3. The copper marble head must be exposed to the outside of the hole for use.

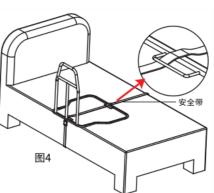


Figure 4. Put the seat belt through the bed and the armrest, tie the armrest to the bed, and adjust the length of the seat belt.

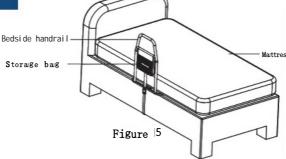


Figure 5. Press the mattress on the handrail of the bed and fill the storage bag to complete the installation work.

Tips: Please check whether the product is damaged or unsecured before use.