




 **Website: <https://elder-careemporium.com/>**

→ "Tap to view product guides"

 **WhatsApp: +8619878772118**



→ "Scan with your phone camera to chat instantly."

 **Email: [info@elder-careemporium.com](mailto:info@elder-careemporium.com)**

**Need help? Contact us anytime!**

## Rollator User Manual

Warm Reminder:

For your safety, please read this manual carefully before use!



Scan to watch the  
installation video

Dear User,

- Thank you for purchasing our product.
- Before using it, please read the user manual carefully to ensure proper use and understanding.
- After reading, please keep this manual in a safe place for future reference.

## PRODUCT STRUCTURE/SPECIFICATIONS

Manual Measurement Tolerance: 1-5 cm

Product Model: YC9102AH

Dimensions: 67 × 62 × (112-124.5) cm

Weight Capacity: 100 kg



1. Open the package and take out the product.



2. Unfold the main frame (as shown in the image above).



3. Remove the locking pin from the rear wheels, align the holes, and insert the pin.



4. Align the holes of the front wheels with the main frame, insert the triangular knob screw, and tighten it.



5. Press down on the seat with both hands to lock it into place.



6. Attach the armrest to the main frame, press the spring button, insert it into the frame support tube, and adjust to the appropriate position.



7. Hang the storage bag on the front of the rollator and secure it to the frame using Velcro straps.




8. Assembly is complete.

## ARMREST ADJUSTMENT / FOLDING INSTRUCTIONS



Armrest Adjustment: Loosen the screw, press the spring button, adjust the handle to the desired position, and then tighten the knob screw.

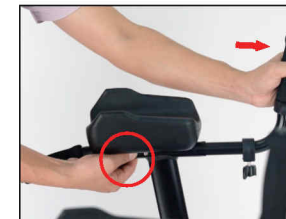

Folding: Pull up the handle strap on the seat to fold the rollator.