




 **Website:** <https://elder-careemporium.com/>

→ "Tap to view product guides"

 **WhatsApp:** +8619878772118



→ "Scan with your phone camera to chat instantly."

 **Email:** info@elder-careemporium.com

Need help? Contact us anytime!

Rollator User Manual

Warm Reminder:

For your safety, please read this manual carefully before use!



Scan to watch the
installation video

Dear User,

- Thank you for purchasing our product.
- Before using it, please read the user manual carefully to ensure proper use and understanding.
- After reading, please keep this manual in a safe place for future reference.

PRODUCT STRUCTURE/SPECIFICATIONS

Manual Measurement Tolerance: 1-5 cm

Model: YC9102H

Dimensions: 66 × 62 × (80-92) cm

Weight Capacity: 100 kg



1. Open the package and take out the product.



2. Unfold the main frame (as shown in the image).



3. Remove the locking pin from the rear wheel, align it with the hole, and insert the pin.



4. Align the front wheel holes with the main frame holes, insert the triangular screw, and tighten it.



5. Press down on the seat with both hands to secure the locking mechanism underneath.



6. Hang the storage bag at the front of the rollator and secure it to the frame using the Velcro straps.



7. Assembly is complete.

FOLDING INSTRUCTIONS



Lift the strap on the seat cushion to fold the rollator.

