


Before using the handrail, please read the instruction manual carefully and use and understand correctly.

Please keep this manual properly after reading, so that you can consult and refer to it at any time.

■YC1215 is a folding handrail suitable for toilet, bathroom, locker room, nursing home, family and other places, the product is made of aluminum alloy, waterproof, no rust, easy to fold, save space, easy to use.


SIZE

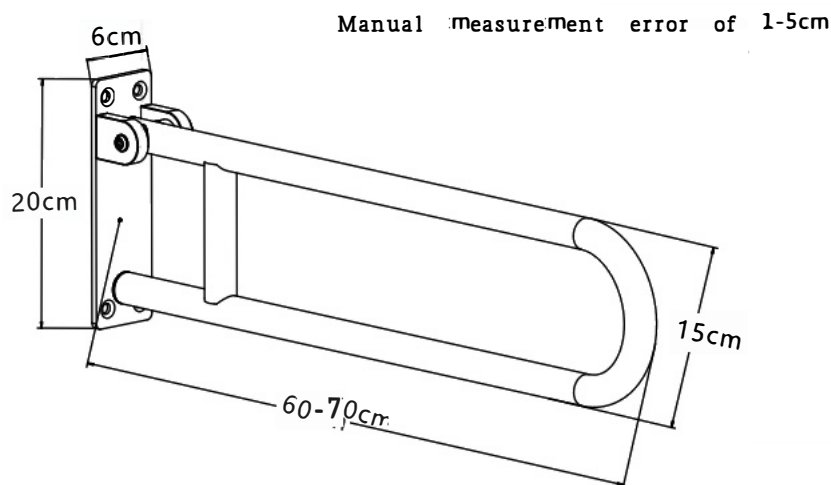

Prepare tools before installation



Electric percussion drill

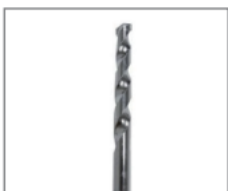

8mm impact bit
Glass bit for ceramic tile wall



Tape measure





Marker pen

Installation precautions:


Ensure that the installation wall is secure before installation.

Walls can be installed: concrete, solid brick, natural dense stone, reinforced wall and other load-bearing walls;


Can not be installed wall: gypsum wall, gray sand brick, thin hollow sand wall, wood panel wall and other low bearing walls.

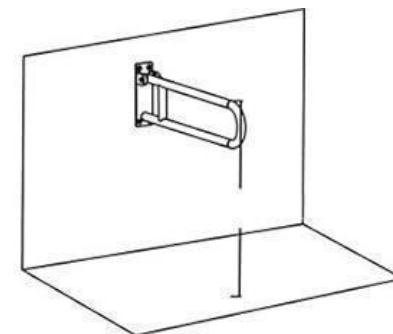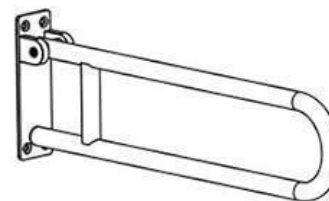


Place to a suitable height

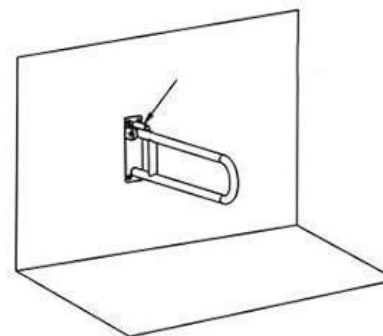


Mark the position of the hole with a marker

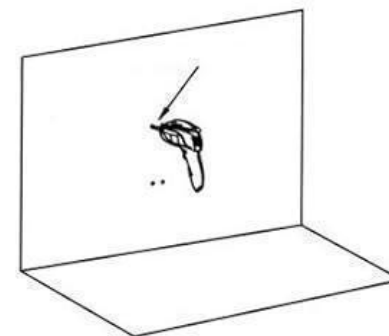

Drilling, the depth is not less than 50mm

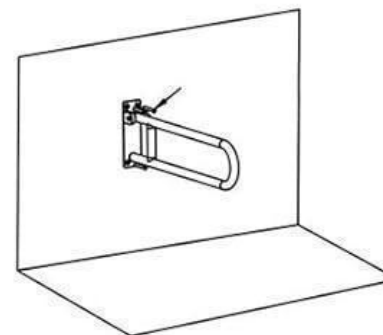

Put in the expansion screw

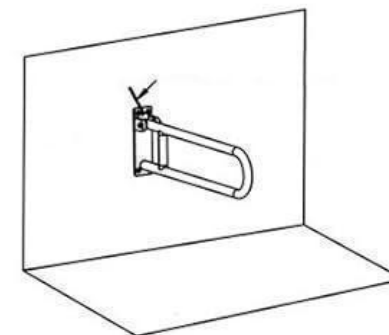

Tighten the screws with a hex wrench